

PULL THRU™ CASSETTE

FEEL THE CLEAN


1.0-1.2™ 1.4-2.6™ 2.8-5.0™ balloon™
PULL THRU™ **PULL THRU™** **PULL THRU™** **PULL THRU™**
 FEEL THE CLEAN FEEL THE CLEAN FEEL THE CLEAN FEEL THE CLEAN

F R Galantai Manufacturing Co. Ltd
 11 Bancroft Crescent, Glendene
 Auckland, 0602
 New Zealand
 ISO 13485:2016 Certified Site

EC REP Advena Ltd.
 Tower Business Centre, 2nd Flr.,
 Tower Street, Swatar,
 BKR 4013 Malta

A

Open Flap on rear of empty Cassette

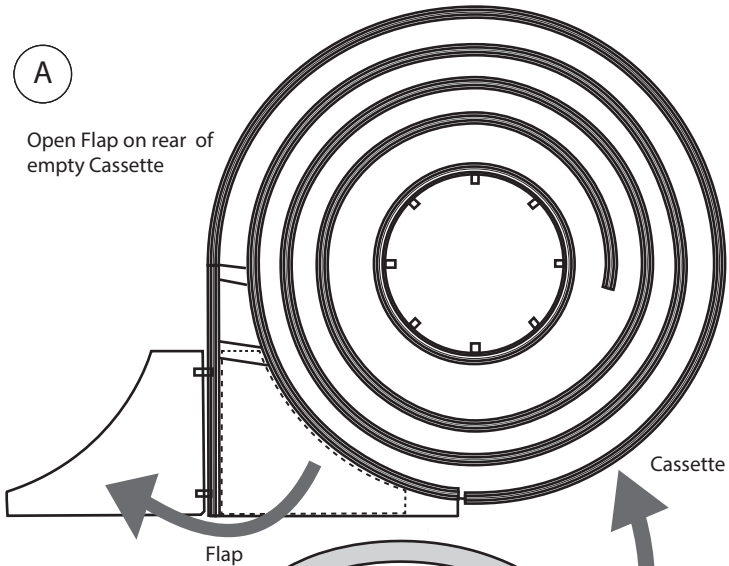



B

Load PULL THRU™
 Cleaning Brush refill
 sleeve into Cassette




C

D

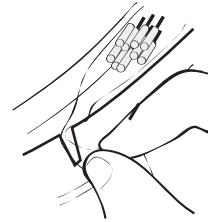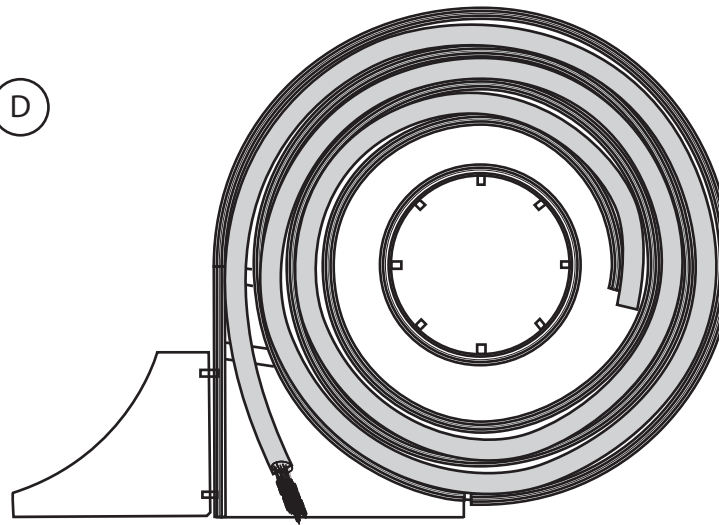

Secure the closed end of the pack by inserting the sleeve through the middle of the slot


E

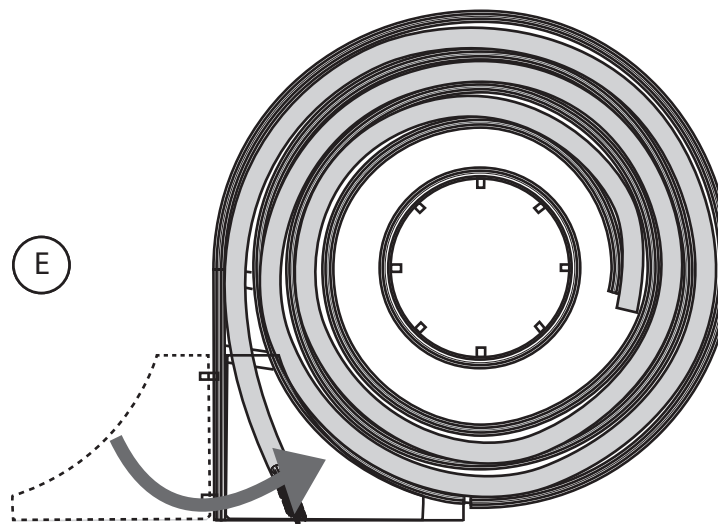


Close Flap

F

Align front of Cassette with Bracket

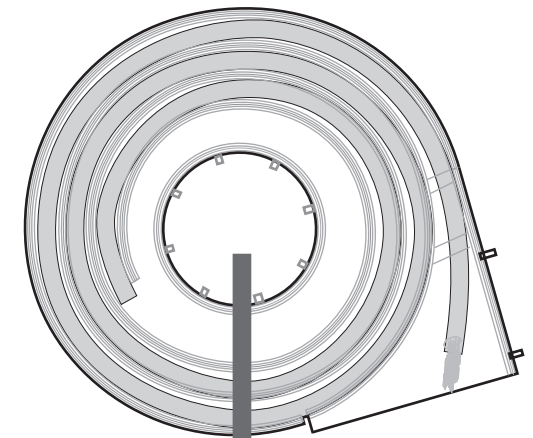


Rotate Cassette to lock into place

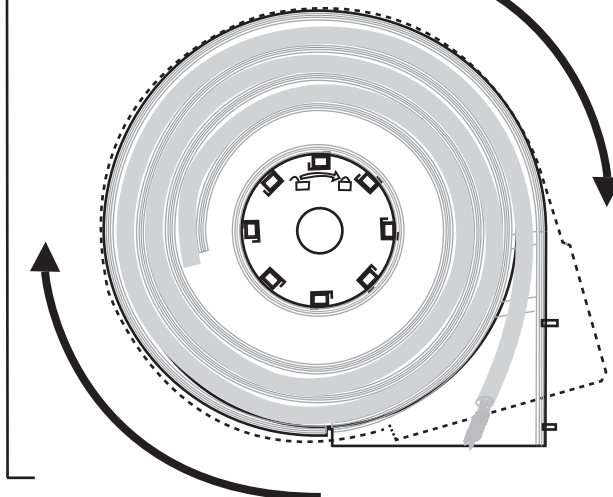


Two methods to attach Bracket


- 1 Screw on surface
- 2 Adhesive

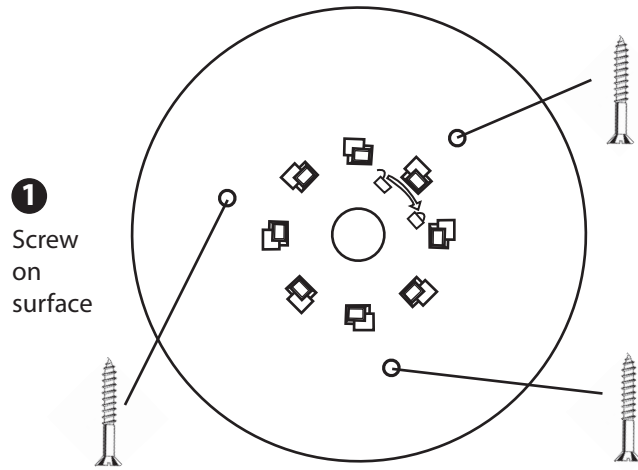

Wall or shelf

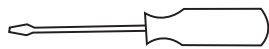


2 Adhesive

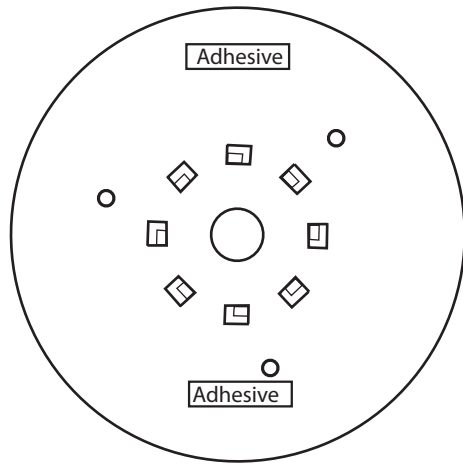


2A

Clean surface Bracket to be adhered to

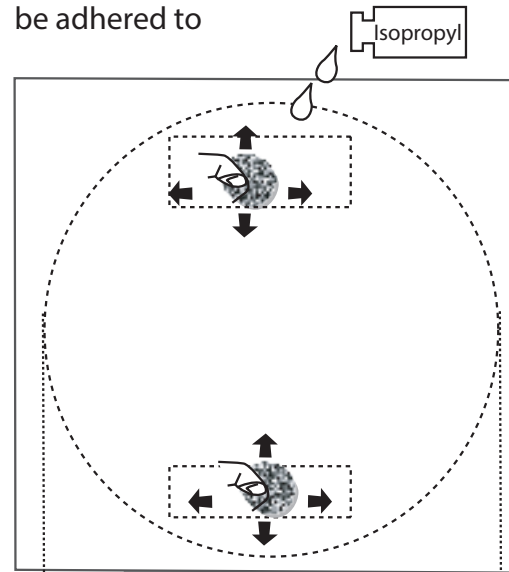

2B

Remove adhesive backing

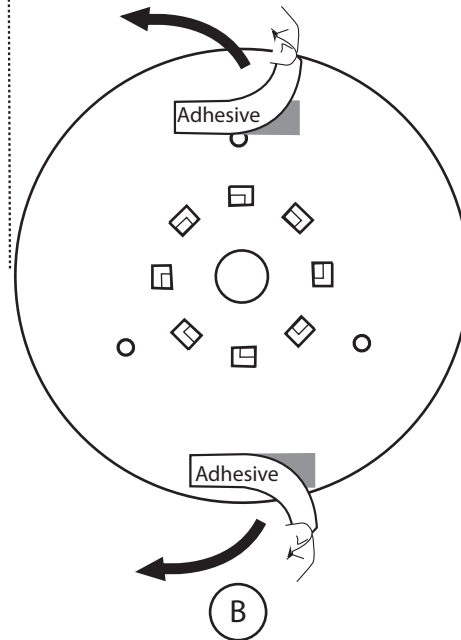

2C

